

DISCOVER THE **OXY** EFFECT - NOW ALSO IN **XL**

Incidin™ OxyFoam & OxyWipe

Ready-to-use cleaner and disinfectant with Hi-speed H₂O₂™

The advanced formulation of Incidin™ OxyFoam & Incidin™ OxyWipe provides the benefits of a patented* hydrogen peroxide solution with broad spectrum efficacy.

They are effective against a wide range of pathogens and leave no active residues due to their rapid evaporation rate.

Incidin™ OxyFoam & OxyWipe provide:

- ▲ Hi-speed H₂O₂™
- ▲ Active ingredient decomposing into water and oxygen
- ▲ Excellent material compatibility



Incidin™ OxyFoam & OxyWipe

Ready-to-use cleaner and disinfectant with Hi-speed H₂O₂™

15 SEC¹

Bactericidal
(EN 13727 / 16615)

15 SEC¹

Virucidal against
enveloped viruses

15 MIN¹

**Noro
virucidal**

Features	Benefits
EFFECTIVE	Broad spectrum efficacy Short contact times
SAFE	Active ingredient decomposes into water and oxygen. Excellent material compatibility
CONVENIENT	Cleans & disinfects in one step Excellent material compatibility

MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS


EFFICACY UNDILUTED	OxyFoam CONTACT TIME	OxyWipe CONTACT TIME
Disinfection of surfaces acc. to VAH method 2015 (incl. EN 16615), dirty conditions		
Bactericidal, yeasticidal	5 min	15 min
Fungicidal	5 min	15 min
Mycobactericidal	15 min	60 min
Tested acc. RKI recommendation 01/2004*		
Effective against enveloped viruses acc. to RKI recommendation 01/2004*	15 sec	30 sec
Adeno*	1 min	15 min
Polyoma SV40*	30 sec	30 sec
Rota*	30 sec	30 sec
Euro-Norms (clean & dirty conditions)		
EN 13727 (bactericidal)	15 sec	1 min
EN 13624 (yeasticidal)	1 min	15 min
EN 13624 (fungicidal)	5 min	15 min
EN 14348 (tuberculocidal, mycobactericidal)	5 min	30 min
EN 14476 Adeno	30 sec	1 min
EN 14476 (limited spectrum efficacy)	15 min	60 min (dirty conditions)
EN 14476 Noro	15 min	60 min (dirty conditions)
Euro-Norms (dirty conditions) - 4-field-test		
EN 16615 (bactericidal)	15 sec	5 min
EN 16615 (yeasticidal)	15 sec	1 min
EN 16615 (fungicidal)	5 min	5 min
EN 16615 (tuberculocidal, mycobactericidal)	15 min	60 min

* DVV 2015 (suspension test)

INCIDIN™ OXYFOAM INSTRUCTIONS FOR USE



- 1 Wet surface
- 2 Wipe the surface ensuring it is completely wet
- 3 Let dry

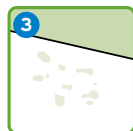

- 1 Apply to clean wipe
- 2 Wipe the surface ensuring it is completely wet
- 3 Let dry

EUROPEAN HEADQUARTERS: ECOLAB EUROPE GMBH

Richtstrasse 7
8304 Wallisellen
Switzerland
+41(0) 44-877-2001
www.ecolab.com



ECOLAB DEUTSCHLAND GMBH

Ecolab-Allee 1
40789 Monheim am Rhein
+49 (0) 2173-599-1900

CE 0297

EFFECTIVE.
SAFE.
CONVENIENT.

APPLICATION AREA

Fast acting, sporicidal ready-to-use surface cleaning and disinfection

- ▲ acc. Directive 93/42/EEC: for medical devices.
- ▲ Biocide: for general surfaces.



INSTRUCTIONS FOR USE

Incidin™ OxyFoam: Wet surface or apply to clean wipe. Wipe the surface, ensuring the surface is completely wet for the contact time. Or use product undiluted on a clean wipe, wipe the surface and let dry. Do not rinse.

Incidin™ OxyWipe: Take single wipe. Wet surface thoroughly, ensuring contact with the entire surface. Let it dry and do not rinse. Close lid after use.

The use of a cleaning and disinfection product does not replace regular cleaning. Not for terminal disinfection of invasive medical devices. For reprocessing of medical devices always observe manufacturers' instructions for use. Do not use on surfaces sensitive to oxidative agents such as marble, copper or brass.

INGREDIENTS

100 g contains the following active:
Incidin™ OxyFoam: 1.5 g
Hydrogen peroxide
Incidin™ OxyWipe: 1.0 g
Hydrogen peroxide.

PACKAGING AND ORDERING INFORMATION

PRODUCT	OUTER SIZE	ORDER CODE
Incidin™ OxyFoam	6 x 750ml	
Incidin™ OxyWipe	6 x 100 wipes (20 x 20 cm)	
Incidin™ OxyWipe XL	6 x 100 wipes (25 x 37 cm)	
wall mounted dispenser for Incidin™ OxyWipe	6 x 1 pcs	10033877

STORAGE

Product should be stored in the original container below 25 °C in a cool, dry location out of direct sunlight.

Shelf life after first opening:
Incidin™ OxyFoam: End of expiry date
Incidin™ OxyWipe: 3 month

References

1. Incidin™ OxyFoam (EN 13727 / EN 16615 / EN 14476)

USE BIOCIDES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE. FOR PROFESSIONAL USE ONLY.

ECOLAB®